

K3Pro® Conventional Rapid Surgical Protocol

using Surgical Steel Drills



Argon Dental USA

Platform	Implant Diameter	Round Bur	Starter Drill	Crestal / Subcrestal Pilot Drills		Implant Drills (Surgical Steel)						Tap	Sinker	Implant Inserter	Cover Screws	
				2.0 mm / 2.5 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm						
			800 rpm	800 rpm	600 rpm	50-600 rpm	50-600 rpm	50-600 rpm	50-600 rpm	50-600 rpm	50-600 rpm	Max. torque 45 Ncm 20 rpm	50 rpm	2mm Platform: Max. torque 35 Ncm 3mm/Short Platform: Max. torque 45 Ncm	Max. torque 5 Ncm	
	3.0															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005						GS_3.0	K_3.0	WSEL_1.4 (Max. torque 35 Ncm)	VSK2K	
	3.5															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005	Rapid: KKB3.5/18.005 Sure/Short: KZB3.5/18.005						GS_3.5	K_3.5	WSEL_1.4 (Max. torque 35 Ncm)	VSK2K
	4.0															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005	Rapid: KKB3.5/18.005 Sure/Short: KZB3.5/18.005	Rapid: KKB4.0/18.005 Sure/Short: KZB4.0/18.005					GS_4.0	K_4.0	WSEL_1.6 (Max. torque 45 Ncm)	VSK3K VSK3K_S
	4.5															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005	Rapid: KKB3.5/18.005 Sure/Short: KZB3.5/18.005	Rapid: KKB4.0/18.005 Sure/Short: KZB4.0/18.005	Rapid: KKB4.5/18.005 Sure/Short: KZB4.5/18.005				GS_4.5	K_4.5	WSEL_1.6 (Max. torque 45 Ncm)	VSK3K VSK3K_S
	5.0															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005	Rapid: KKB3.5/18.005 Sure/Short: KZB3.5/18.005	Rapid: KKB4.0/18.005 Sure/Short: KZB4.0/18.005	Rapid: KKB4.5/18.005 Sure/Short: KZB4.5/18.005	Rapid: KKB5.0/18.005 Sure/Short: KZB5.0/18.005			GS_5.0	K_5.0	WSEL_1.6 (Max. torque 45 Ncm)	VSK3K VSK3K_S
	6.0															
			KB1.9K	KB.STLD	KB2.0 KB2.5	Rapid: KKB3.0/18.005 Sure/Short: KZB3.0/18.005	Rapid: KKB3.5/18.005 Sure/Short: KZB3.5/18.005	Rapid: KKB4.0/18.005 Sure/Short: KZB4.0/18.005	Rapid: KKB4.5/18.005 Sure/Short: KZB4.5/18.005	Rapid: KKB5.0/18.005 Sure/Short: KZB5.0/18.005	Rapid: KKB6.0/12.005 Sure/Short: KZB6.0/12.005		GS_6.0	K_6.0	WSEL_1.6 (Max. torque 45 Ncm)	VSK3K VSK3K_S



QUALITY & DESIGN
MADE IN GERMANY

