

K3Pro™ Precision Guided Surgical Protocol using Titanium Drills



Example of a Surgical Protocol sheet included with your case.

Argon Dental USA
ID: 1 013002020

IMPLANT: 4.5 x 11.0 Rapid (Insertor Depth - 7mm) TOOTH# 21

PATIENT: DOCTOR:

Punch or Trepan 50 rpm

Starter Drill 200 rpm

Pilot Drill 200 rpm

Cortical Drill 100 rpm

Final Drills 100 rpm

Top 20 - 40 rpm Torque Max 45 ncm

Implant Insertor Contra/Angle Hand/Ratchet 20 rpm, Torque Max 45 ncm


DISCLAIMER: The purpose of this Argon K3Pro Surgical Protocol is to provide support and guidance to you and your staff. You should always defer to your own professional judgement. In the event you have any questions or other doubts, you should seek advice. Nothing in this protocol is intended to create nor does it create any enforceable guarantees, promises, rights, remedies, entitlements and/or obligations by and between Argon Dental USA LLC or Implant Solutions LLC and yourself.

K3Pro® Fully Guided
Precision Surgical Kits

QUALITY & DESIGN
MADE IN GERMANY



3.0 - 3.5mm Kit

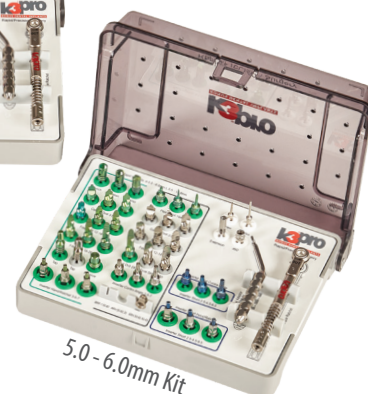

5.0 - 6.0mm Kit

- Find the "K3Pro Surgical Protocol" sheet enclosed with your case. Starting with Stage 1, follow the step-by-step protocol which includes rpm and torque recommendations by the manufacturer.

Argon Dental USA
ID: 1 013002020

IMPLANT: 4.5 x 11.0 Rapid (Insertor Depth - 7mm) TOOTH# 21

PATIENT: DOCTOR:

Punch or Trepan 50 rpm

Starter Drill 200 rpm

Pilot Drill 200 rpm

Cortical Drill 100 rpm

Final Drills 100 rpm

Top 20 - 40 rpm Torque Max 45 ncm

Implant Insertor Contra/Angle Hand/Ratchet 20 rpm, Torque Max 45 ncm



- Continue with the sequential drills noted in Stage 2 to complete the osteotomy.

Argon Dental USA
ID: 1 013002020

IMPLANT: 4.5 x 11.0 Rapid (Insertor Depth - 7mm) TOOTH# 21

PATIENT: DOCTOR:

Final Drills 100 rpm

Top 20 - 40 rpm Torque Max 45 ncm

Implant Insertor Contra/Angle Hand/Ratchet 20 rpm, Torque Max 45 ncm



- Complete the guided surgery process following the protocol steps for Stage 3, placing the implant directly through the surgical guide to ensure proper placement.

Argon Dental USA
ID: 1 013002020

IMPLANT: 4.5 x 11.0 Rapid (Insertor Depth - 7mm) TOOTH# 21

PATIENT: DOCTOR:

Top 20 - 40 rpm Torque Max 45 ncm

Implant Insertor Contra/Angle Hand/Ratchet 20 rpm, Torque Max 45 ncm

